Hallufix®

INNOVATIVE ORTHOTIC SOLUTIONS

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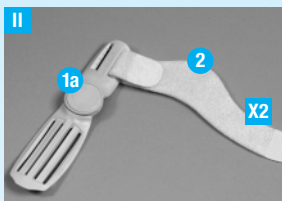
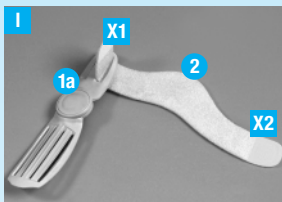
For effective correction of the deformity of the big toe (bunion)

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Threading toe bandage into the splint

Place the toe strap in front of you as depicted in Fig. I. The hook-and-loop end (X1) of the toe strap (2) is drawn from below through the right opening of the toe blade (1a) and is fastened to the toe strap (2) → (Fig. II). The second hook-and-loop end (X2) is later also drawn through the left opening of the toe blade (1a) and fastened to the toe strap (2) (Fig. II).

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


Threading the mid-foot strap into the splint

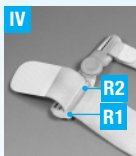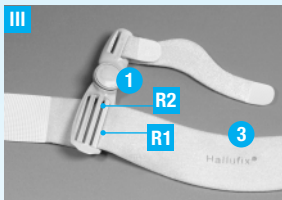Place the splint (1) and the mid-foot strap (3) in front of you as shown in Fig. III. The mid-foot strap is drawn from above through the reversing slit (R1) and then from below through the reversing slit (R2) (Fig. IV) and fastened to the mid-foot strap (3) (Fig. V). The second end of the mid-foot strap (3) is drawn from above through the reversing slit (L1), fed back through slit (L2) and then fastened to (3) (Fig. VI).

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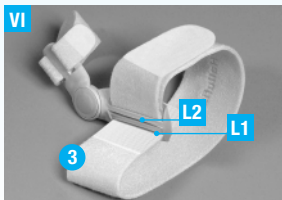
Foam side cushion

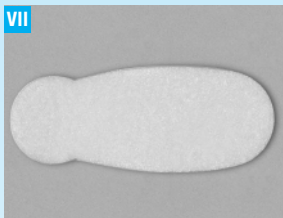The side cushion (Fig. VII), as delivered in the original packaging, is fastened securely to the splint with hook-and-loop. Should you have to fasten it yourself (e.g. after washing), please ensure that the side cushion is placed on the long part of the splint; the shape of the cushion fits the shape of the splint.

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Inserting the mid-foot pelotte

The mid-foot pelotte is inserted on the underside of the foot with the hook-and-loop towards the strap and is fitted in the arch of the foot in order to lift the metatarsal bones. It has a very important function while walking and should always be used except when the shoes contain a foot or arch support.

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Important advice

Lifecircle average of bandages and foam cushion is max.3 months, depending on frequency of wearing, duly usage and care. In case of heavy duty and often cleaning substitution in shorter periods is advised.

 Handwäsche in milder Seifenlauge lauwarm, ohne Weichspüler, **Delicates (laundry bag)**

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Hallux Valgus (bunion)

Mode of action:

- corrects the deformity of the big toe
- preventive and post-operative
- shortens rehab time
- reduces the risk of arthritis
- keeps your feet flexible
- fits in comfortable shoes

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Hallufix® is a registered trade mark.
International patents issued.

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Hallufix AG